

RESUSCITATION TROLLEY DAILY CHECKLIST (Mindray D3 Defibrillator)

Week Commencing Ward/Department

	Mon	Tues	Wed	Thur	Fri	Sat	Sun
Date:							
Box of Gloves							
Empty Sharps Bin							
Bag Valve Mask including Size M adult Face Mask							
15L Reservoir Mask							
Oxygen Tubing							
Pocket Mask							
End Tidal CO2 Line							
Top of trolley appears clean and tidy							
Oxygen and Suction							
Oxygen cylinder, contents above ½							
Suction available & working							
Defibrillator Check							
Mindray D3 Defibrillator plugged in, clean and dust free. No visible damage.							
Battery installed with at least 60% of battery capacity							
2 sets of defibrillator pads unopened in sealed packet (in date)							
ECG leads present, undamaged, and connected securely.							
ECG electrodes in date (1 packet)							
Recorder paper present							
Check Auto check completed and illuminated green tick visible If red cross , please complete User Test and report to EMBE if unresolved.							
Complete User Test: - Routine test - Energy deliver test	Weekly & after use						
Trolley sealed ID Number							
Nurse signature							

To comply with ELHT Medical Devices Policy this record must be kept for 2 years
 All learning for this device is situated on the Learning Hub via the QR Code

